SmartBand™ Ligation Kit Instructions for Use

ENGLISH

Intended Use

This device is used to endoscopically ligate esophageal varices at or above the gastroesophageal junction and to ligate internal hemorrhoids.

Device Description

The SmartBand Ligation Kit consists of a ligation handle, loading device, connector tube and a cylindrical barrel with 6 latex bands. The deployment cord separates each band circumferentially. The deployment cord is passed through the working channel of the endoscope using the SmartBand loading device and then assembled to the SmartBand ligation handle.

Contraindications

Patients with a known latex hypersensitivity.

Those specific to primary esophagogastroduodenoscopy procedures that would prevent the physician from gaining access to the treatment site.

Those specific to esophageal ligation include, but are not limited to: coagulopathy, cricopharyngeal or esophageal narrowing or stricture, tortuous esophagus, diverticula, known or suspected esophageal perforation, asymptomatic rings or webs.

Those specific to hemorrhoidal ligation include, but are not limited to: severe inflammatory bowel disease or portal hypertension.

Potential Complications

Those associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Those that can occur with esophageal include, but are not limited to: retrosternal pain, nausea, laryngeal laceration, esophageal perforation, stricture formation, obstruction or ulceration.

Those that can occur with hemorrhoidal ligation include, but are not limited to: severe pain, bleeding, urinary symptoms, swelling and edema, tissue ulceration, bands slipping off. Hemorrhoidal ligation may result in severe pain if the procedure is performed below the dentate line.

Precautions

Refer to package label for minimum channel size for this device.

Band ligation may not be effective when applied to small varices.

Esophageal ligation devices are not intended for ligation of varices below the gastroesophageal junction.

Passing endoscope over a previously placed band may dislodge band.

Prior to assembling device, routine endoscopic examination is recommended to confirm diagnosis requiring treatment of esophageal varices or internal hemorrhoids.

It is vital that the integrity of the working channel is intact as grooves or other obstructions in the channel can potentially cause the string to catch, resulting in band deployment difficulty. Use of an endoscope in a sound state of repair is a prerequisite for a successful multi-band ligation procedure.

Reuse, reprocessing or sterilization may create a risk of contamination of the device or cause patient infection, or cross-infection, including, but not limited to, transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Notes

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure.

This device is supplied non-sterile

Do not use this device for any purpose other than stated intended use.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If abnormality is detected that would prohibit proper working condition, do not use. Please notify Intelligent Endoscopy for return authorization. Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

Cautions

Federal U.S. law restricts this device to sale by or on the order of a physician.

System Preparation

- Insert ligation handle onto endoscope accessory port as follows:
 - **Olympus/Fujifilm endoscopes:** Insert handle stem through cap and into accessory channel. (fig 1a)
 - **Pentax endoscopes without luer lock connection:** Insert handle stem through cap and into accessory channel. (fig 1a)
 - **Pentax endoscopes with luer lock connection:** Remove adapter from inlay card and attach directly to accessory port luer connection. **Note:** No cap required on accessory port. Then insert handle stem into adapter to form a secure fit. (fig 1b and fig 1c)
- Introduce loading device through center of ligation handle and advance until tip of loading device exits endoscope. (fig 2) **Note:** Either end of the loading device may be introduced into the accessory channel.
- Attach deployment cord loop to hook feature of loading device. (fig 3)
- Withdraw loading device and deployment cord through endoscope and ligation handle.
- Place barrel onto tip of endoscope and advance barrel until the scope tabs stop barrel. (fig 4) **Note:** Keep the deployment cord clear of the barrel and endoscope, so that it does not become caught.
- Attach deployment cord loop onto ligation handle hook feature. (fig 5)
- With deployment cord secure on ligation handle, rotate ligation handle slowly to remove the extra slack in the deployment cord. (fig 6) **Note:** Putting too much tension on the deployment cord can cause premature band deployment.
- To optimize endoscopic view, rotate barrel to change the deployment cord position. **Note:** With each band deployment, endoscopic view is optimized.
- Lubricate endoscope and exterior portion of barrel. **Caution:** Do not place lubricant inside of barrel. **Caution:** Do not apply alcohol to device.

The device is fully assembled and ready for use.

Instructions for Ligation of Esophageal Varices

- With device assembled onto endoscope, introduce endoscope into esophagus.
- Select varix and suction into barrel.
- While maintaining suction, turn handle forward to deploy band. Band release will be felt indicating band deployment.
- Release suction and insufflate air, then gently withdraw endoscope to release ligated varix. **Note:** If irrigation is needed to clear endoscopic view, attach connector tube to dedicated flush port (fig 7a) on ligation handle and inject sterile water using a 60cc syringe. (fig 7b)
- Repeat steps 2 – 4 as needed.
- If more bands are required, remove endoscope and use a SmartBand ligation pack.

Instructions for Ligation of Internal Hemorrhoids

- With device assembled, introduce endoscope into rectum.
- Retroflex endoscope and select internal hemorrhoid.
- Suction internal hemorrhoid into barrel and while maintaining suction, turn handle forward to deploy band. Band release will be felt indicating band deployment.
- Release suction and insufflate air. Then, gently advance endoscope to release ligated internal hemorrhoid. **Note:** If irrigation is needed to clear endoscopic view, attach connector tube to dedicated flush port (fig 7a) on ligation handle and inject sterile water using a 60cc syringe. (fig 7b)
- Repeat steps 2 – 4 as needed.

Removing the SmartBand Ligation Device from the Endoscope

All bands have been deployed:

- Remove handle and attached deployment cord from accessory channel.
- Remove barrel from endoscope tip.

Bands remain on barrel:

- Rotate handle to deploy remaining bands into biohazardous medical waste container.
- Remove handle and attached deployment cord from accessory channel.
- Remove barrel from endoscope tip.

Dispose of the device per institutional guidelines for biohazardous medical waste.

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